

Hettich EBA 20 Home Haemodialysis Centrifuge Set

Your Kit Includes...



1 x Centrifuge



1 x Carry Case



2 x Blood Tube Adapters
(Tubes Not Included)



6 x Fillers (Pre-Inserted into Centrifuge)



1 x Mains Lead

Hettich EBA 20 Home Haemodialysis Centrifuge Set

How To Use Your Centrifuge

Step 1—After taking the centrifuge and all parts out of the box, plug in the centrifuge and turn on using the on/off switch on the back of the centrifuge.



Step 2 — Press the open lid button (See Picture Left) and lift the lid.



Step 3 — Ensure the pre-inserted fillers are not removed but pushed into the holes as far as possible and remain spaced as per picture right.



Step 4 — Place the two blood tube adapters in the empty slots, place your blood tube in one of these adapters and a balance tube in the other (See Picture Left). The blood tubes will protrude from the adapter by approx. 2.5cm.

Step 5 — Close the lid and lift to ensure lid is locked. Press the Start Button (The centrifuge is pre-set to correct speed and time).



Step 6— When the cycle has finished and the rotor has come to a standstill press the open lid button and lift the lid to remove tubes.